KEMOT®

Alcohol breath tester MIE0217

User Manual

English 1-10

Polski 11-20

Română 21-30

Deutsch 31-40

Table of Contents

Introduction	4
Correct measurement	4
Description of parts	4
Power supply	5
Operation	5
Accessories	8
Error and troubles	8
Important notes	9
Specifications	9
Calibration	10
Precautions	10


Introduction

The device is intended to measure alcohol in the human breath. Measurements obtained by this device are used in the diagnosis of alcohol intoxication.

Correct measurement


- Before testing, wait at least 30 minutes after your last drinking.
- Only use the device in a temperature 10°C ~ 40°C or 50°F ~ 104°F.

Description of parts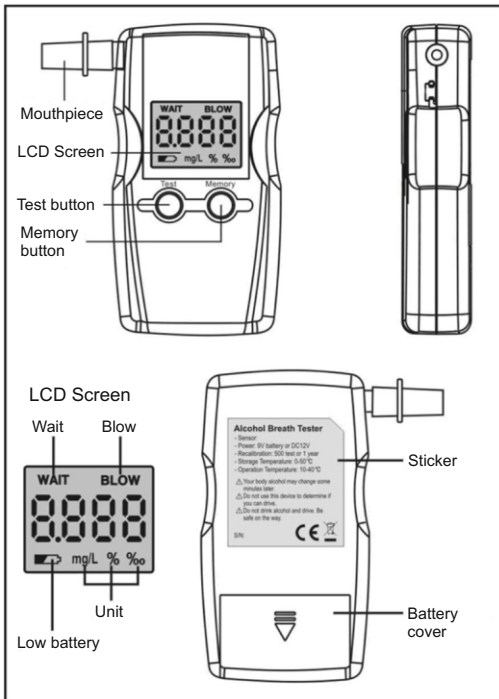

Power Supply


- Slide back the battery cover on the back of the device.
- Insert 1 piece 9V alkaline battery, making sure that the “+/-”symbols are correctly aligned.
- If the battery voltage is too low, the unit may display a battery symbol „” and the device may not operate correctly. You need to change a new battery.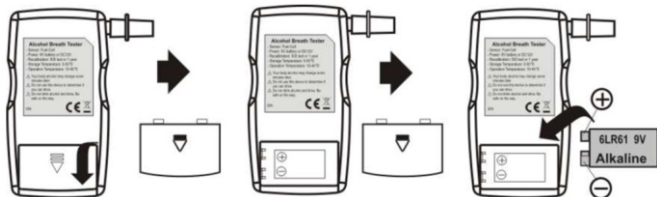


Operation

(Test mode)

- Insert one of the mouthpieces into the 'blow in' hole.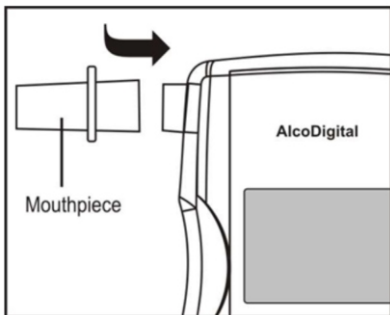


- Press and hold the TEST button until power on. Release it after power on.

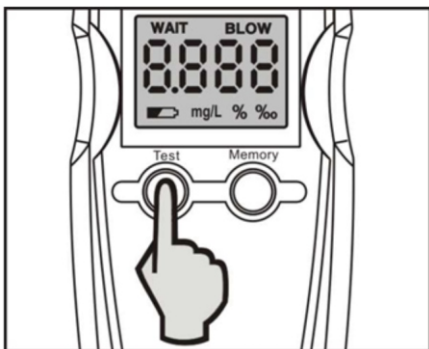- The WAIT displays on the screen and countdown 10, 9, and 8... to zero.

- The BLOW displays on the screen, and displays "0" with flashing. Take a deep breath, blowing steady into the mouthpiece until beeping stops (about 5 seconds).

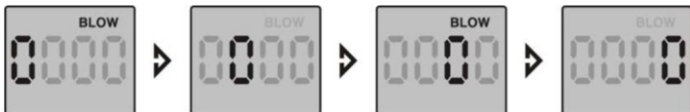
- Your test result will be displayed on the screen for 20 seconds.

(Memory mode)

- After tested, the device displays the test result for 20 seconds, press the "Memory" button to check your previous tested records.

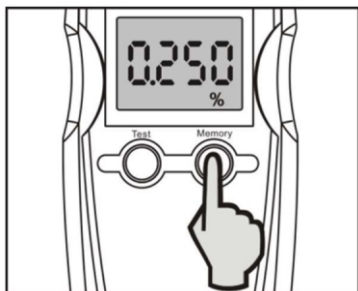
- The device will auto off after stopping operation.
- The memory button is only available on this mode.

Accessories

- Mouthpieces
- Manual

Error and troubles

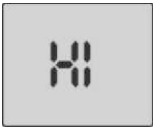Error message or troubles	Description	Remarks
No display	-Press and hold the TEST button until power on. -Check the battery	
Auto Off	-Low battery -Over the period of test stage	
	-Low battery indication -Need to change new battery -Use alkaline battery	
Alarm sound	-Over alcohol display range with "HI" indication -Blowing time after warm up	
HI	-The BAC above the display range. -Display range: 0.000-0.80 g/L BAC	

Important notes

- Before testing, wait at least 30 minutes after your last drinking.
- After countdown, blow immediately and blow until beeping stop.
- The memory recalls function only available after tested and 20 seconds.
- Only use the device in a temperature 10 - 40°C or 50 - 104°F.
- Using 9V alkaline battery only.
- Wait 3 minutes for retest.
- Recalibration: each 12 month or 200 tested times.
- Never store the device in a corrosive gas room.
- Never store the device under sun exposure area.
- Avoid dropping or subjecting the device to a strong shock.

Specifications

- Semiconductor Sensor.
- Recalibration after 200 test time or 12 months.
- Detection range: 0.000-2.000g/L BAC.
- Display range: 0.000-0.800 g/L BAC.
- Show "HI" if above the display range.
- Accuracy: +/-0.15 at 0.50 g/L.
- Memory recall for last 20 tested data.
- 5 mouthpieces included.
- Warm up time: 15 seconds.
- Testing time: 10 seconds.
- Repeat test: no less than 3 minutes.
- Audible sound alarm if above 0.50 g/l BAC
- Indication: 4 digits LCD with status and backlight.
- Power supply: 9V alkaline battery (not included)
- Battery available for standard testing is about 200 times.
- Low battery indication and auto power off function.
- Operation: 10°C - 40°C; Store: 0°C - 50°C
- Size: 120x65x25mm

Calibration

- You should have your alcohol tester calibrated every 12 months or 200 tested times. Failure to do so may cause your readings to be inaccurate.

Calibration Record			
Date First Used		Serial Number	
#	Calibration Date	Remarks	Recalibration by

Precautions

- Your blood alcohol content (BAC) may change some minutes later.
- Do not use this device to determine if you can drive.
- You cannot use the test results of device in court of law or against police.
- For your safety and responsibly, don't drink and drive.

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English
**Correct Disposal of This Product
(Waste Electrical & Electronic Equipment)**

(Applicable in the European Union and other European countries with separate collection systems)

This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Made in China for LECHPOL Zbigniew Leszek, Miętne, 1 Garwolińska Street 08-400 Garwolin.

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